

Caring Spoon

Help People with Hand Tremor to Eat without STRESS..



There are about 60 million people cannot eat normally because of hand tremor due to various reasons, and about 15 million of the hand tremor is due to Parkinson's disease.

Caring Spoon is designed to make life easy of people with hand tremor; to eat without stress. It can preserve their self-care ability of daily living and regain their self-esteem with confidence.







Made from Medical grade materials.

Tritan[™] covering, BPA free, clarity, toughness,
and chemical resistance.

 Medical grade silicone, anti-bacterial, ensure a safe eating experience.





Easy to use Automatic mode.

- Switch on and off automatically.
- Automatically turn to sleep mode when not in use.
- Significantly reduce power consumption.







Caring SPOON is designed to help those people with hand tremor to eat without stress, it can preserve their self-care ability of daily

living and enhance their self-confidence.

Comfortable handle design Excellent quality and suitable size

- Total weight 130g, easy to hold.
- Ergonomic design, the design adapted to the natural hands' use habit, easy to hold and also anti-slippery.








The spoon excllently utilize principle of rehabilitation robotics and make dining smoothly for patient with hand tremor.

 The highly sensitive sensors continuously detect movement of spoon handle caused by hand tremor.

 Microprocessor analyze and dive motors to counter-balance the spoon head to reduce offset by hand tremor.

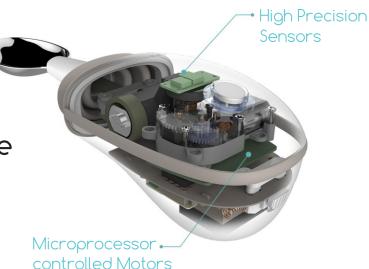



Controlling Range

Caring SPOON is best for people with hand tremor range between 7.0 cm (2.7 in) horizontal and vertical.







Attachments all you need

Easy to change and clean Spoon and Fork head for all your meals. Now Soup and Food both can be enjoyed without stress.







Rechargeable Battery with Docking Charger



- Sustainable Battery.
- Fully charged battery can run three meals a day.
- The maximum use of time is 180 minutes.
- Replaceable battery designed.
- The low battery LED notification.

Connect the attachment to turn it on automatically, enjoy your meal, then simply use the dock to charge.



For Trade Inquiries, Please Contact: Exclusive Authorized Distributor – <u>WayPar</u> GW Trading LLC, Dubai, UAE

Website - www.wayparme.com, Email - info@wayparme.com

